IMPLANT DE RENFORT COMPOSITE DOUBLE FACE
COMPOSIT DOUBLE-SIDED REINFORCEMENT IMPLANT

MICROVAL-Intra

MICROVAL
FRANCE

IMPLANT COMPOSITE DOUBLE FACE COMPOSIT DOUBLE-SIDED IMPLANT

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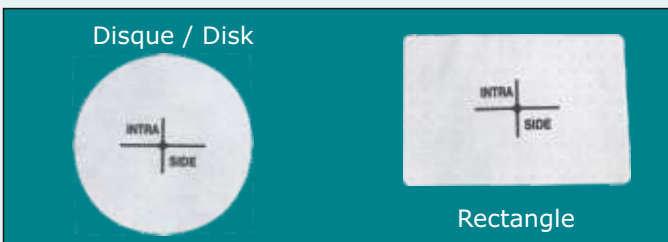
The MICROVAL-INTRA double-sided composit implant is made of 100% non-woven polypropylene film of 50 or 70 g/m² density, impregnated on one side with silicon, non-adhesive to the blowels.

Its parietal side is a microporous structure of thermally bonded polypropylene fibers, for an easier tissues fixation and colonization.

Its visceral side is a sleek silicon layer to avoid viscera and bowels adhesions, creating then a pseudo-peritoneum. To make easier the placement of the implant, the visceral side is identified with one cross and "intra side" words printed on it with black silicone ink.

Macro-perforations on the surface of the MICROVAL-INTRA implant allow a good drainage and avoid seroms.

This 100% biocompatible structure is designed for eventrations, umbilical hernia repairs and wall reinforcement, in intra-peritoneal position. It can be used as well in coelioscopie or open surgery. Use only non-resorbable sutures or staples for its fixation.

These implants exist in different sizes, as listed hereunder. All other different shapes can be supplied upon request. Please do not hesitate to contact us or visit our website.

Références/References		Formes Shapes	Dimensions/Sizes (cm)
50 g/m ²	70 g/m ²		
452015	472015		Rectangle 10 x 15
452517	472517		Rectangle 15 x 17
452025	472025		Rectangle 20 x 25
452030	472030		Rectangle 20 x 30
452012	472012		Disque diametre /Disc diameter 12
452016	472016		Disque diametre /Disc diameter 16
452317	472317		Ovoïde/Ovoid 13/15 x 17
452522	472522		Ovoïde/Ovoid 10/15 x 22
452530	472530		Ovoïde/Ovoid 15/20 x 30
452033	x		Carré/Square 30 x 30

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